## INSTALACE TMRTracker

You have received the CDrom to install TMRTracker on your pc. Before you start you have to make sure that all programs are closed.

TO INSTALL THE PROGRAM, COMPLETE THE FOLLOWING STEPS:

1. 1. Insert the CDrom into the CDrom drive of your pc. If the setup does not start automatically, you can click on the Start button on the Taskbar and select Run. Type 'd:\setup.exe'. If your CDrom drive has a different letter, please fill in that one instead of 'd'. Click OK
2. 2. Read the licence agreement and Accept.
3. 3. Fill in the Serial Number and click Next

**Your Serial Number is shipped with the CD. DO NOT LOSE THIS NUMBER! Software license is unique. Please secure for re-installation!**

1. 4. Select your country and click Next 3 times. If you already installed TMR Tracker 3 before, you can select the country with Update behind it.
2. 5. If the System check is successful, press Start. The installation starts.
3. 6. After this you are asked to restart your pc. Accept and click Finish.
4. 7. By doubleclicking the TMRTracker icon on the desktop you can start the program.
5. 8. A screen appears to select a farm. Click on Add Herd and fill in your herds name and number and click OK.
6. 9. The new herd is made. The Superpump may be started.
7. 10. If you start the program you will see it is not registered yet. Choose Help – Registration – Help in the menubar. Here you can find how to register the program. Please do this within 30 days; otherwise you will not be able to enter the program anymore!
8. 11. To be able to exchange data with the DataKey, you need to connect the Docking Station to your pc.

WINDOWS XP:

1. 12. The Wizard New Hardware appears
2. 13. Select: 'No, not this time' en click Next twice, and Finish
3. 14. Repeat step 13.
4. 15. The drivers have been installed, de installation is finished

WINDOWS VISTA:

1. 16. The screen 'New Hardware' appears. You are asked to install leading programs. Click on this and click Next.
2. 17. Click 'Not online' followed by Next. After the drivers have been installed, press Finish.
3. 18. Repeat step 16/17 if prompted again.
4. 19. The drivers have been installed, de installation is finished

You can find the help in the TMRTracker program by clicking on the helpicon.

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